BEHREND HOMECARE® Nagelpilz Stift

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CE 0481
PZN18275217

BEHREND HOMECARE® Fungal Nail Pen

For the treatment of nail fungus on feet and hands

GENERAL ADVICE:

Before using BEHREND HOMECARE® Fungal Nail Pen, read the instructions carefully, even if you have used this medical device before. The information contained in a new pack may have been revised in order to reflect the latest scientific insights and data. Nevertheless, you should use BEHREND HOMECARE® Fungal Nail Pen with care in order to obtain good results. Keep these instructions – you may need to refer to them during treatment.

APPLICATION:

BEHREND HOMECARE® Fungal Nail Pen is a medical device that helps in the treatment of fungal nails (onychomycosis). BEHREND HOMECARE® Fungal Nail Pen has been proven to be effective in the treatment of the moulds that cause fungal nails.*

*Based on in vitro anti-mycotic effectiveness tests conducted on Trichophyton rubrum, Trichophyton mentagrophytes, Epidermophyton floccosum and Candida albicans.

To achieve faster, better results, carefully filing the fungal nail once a week is recommended. If treatment is stopped, the fungal nail will return more quickly. It is therefore important to continue using BEHREND HOMECARE® Fungal Nail Pen until the fungal nail has gone.

As damp environments are one of the causes of fungal nail infections, it is strongly recommended that you wear socks made only of absorbent materials, such as cotton or wool. This is important, as moulds are commonly found in moist environments, where they can multiply. If you have a fungal nail, it is important that you keep your feet dry.

DOSAGE AND USE (for adults only):

BEHREND HOMECARE® Fungal Nail Pen is for external use only. **Carefully twist the back of the pen until a drop of liquid is released.** Apply the product to the affected area twice or three times a day (mornings and evenings, for instance) until the fungal nail has fully grown out. Fungal Nail Pen can be combined with the use of oral antimicrobials.

MODE OF ACTION:

Thanks to a unique combination of ingredients, BEHREND HOMECARE® Fungal Nail Pen forms a layer on the nail plate, which creates an mildly acidic environment unfavourable for the growth and spread of fungi.

SIDE EFFECTS:

Possible side effects includes: erythema, irritation, edema, pain, nail discoloration and onycholysis. Allergic reactions.

USE DURING PREGNANCY AND/OR BREASTFEEDING:

No information is available regarding the use of BEHREND HOMECARE® Fungal Nail Pen during pregnancy or the breastfeeding phase. If you are pregnant or breastfeeding, consult your doctor or pharmacist before using.

WARNINGS:

- For external use only.
- Fungal nail might cause erythema, irritation, edema, pain, nail discoloration and onycholysis.
- Check the list of ingredients to know if you are allergic to any of the ingredients contained in the gel.
- Avoid contact with eyes or mucous membranes.
- Do not apply to open wounds or damaged skin.
- Do not bandage tightly.
- Do not apply heat to the area of application.
- Pregnant or breastfeeding patients should seek advice of health professional before using Fungal nail.
- If you experience allergic reactions, stop using the product.
- Keep out of reach of young children.
- In case of no effect, consult a physician.

DO NOT USE BEHREND HOMECARE® Fungal Nail Pen IF:

You are hypersensitive to any of the ingredients.

COMPOSITION:

Aqua, Ethoxydiglycol, ISK1320, Pentylene Glycol, Alcohol Denat., Tridiceth-9, PEG-40 Hydrogenated Castor Oil, Benzyl Alcohol, Limonene, Glycerin, Natilact, Propylene Glycol, PVP, Xanthan Gum, Eucalyptus Globulus Leaf Oil, Lavandula Hybrida Oil, Linalool, Dimethyl Isosorbide, Citrus Aurantium Dulcis Oil, Pogostemon Cablin Oil, Disodium Phosphate, Geraniol, Citral, Biotin, Coumarin, Citric Acid, Farnesol.

STORAGE INSTRUCTIONS:

Store in a dry and dark place at room temperature (5-25°C). Keep out of the reach of young children.

See the bottom of the pack for the expiry date.

MEDICAL DEVICE

BEHREND HOMECARE® Fungal Nail Pen is a class IIA medical device according to the European medical devices directive no. 93/42/EEC.

Date on which these instructions were last revised: 08-2022 - 074382-V1.3

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